

Product Description

ABENA Slip is our Premium range of fully breathable, anatomically shaped bladder protection briefs for moderate to heavy incontinence. The high performance core with Top-Dry Technology ensures fast absorption and a dry surface even after multiple voids, maintaining skin integrity. A combination of different barriers (visible and invisible) and elastics (curved leg elastic) offer 360 degree leakage protection all around the product. When combined, these functionalities keep the skin in balance and maintains a natural pH skin balance. The odour system minimizes unpleasant smells.

The high quality flexible combi tapes with double tape function ensures an optimum fit and easy repositioning. The wetness indicator with graduation scale guides the user and caretaker to the product with the right absorption level and aids product changes. The dermatological tested products feature the Nordic Eco Label.



Specifications

Base name	Briefs			
Brand	ABENA			
Sub-brand	Slip			
Size	M2			
Color	White			
Features	Blue colourcode			
Material	Breathable textile backsheet (BTBS), ECF fluff, elastane, elasticated combi tape, nonwoven, PE, PP, resin, SAP			
Length/depth	840 mm			
Width	670 mm			
Hip size	70-110 cm			
Tolerance	+/- 10%			
Absorption Rothwell	2600 ml			
Gender	Unisex			
Quality level	Premium			
Certifications	CE, MD, Nordic Swan Ecolabel, Dermatologically tested, UKCA			
CE Class (Medical Devices)	Class I			
Product or test standards	ISO 11948-1, ISO 13485, ISO 14001, ISO 9001			
Directives, regulations and acts	MDR (EU) 2017/745			
Safety Instructions and Warnings	Keep out of reach of children.			



Because of the products composition it has an expiry date from the production date on:	5 years		
Storage Instructions	Store dry, room temperature and no direct sunlight.		
Product Disposal Instructions	Dispose with household waste.		
Packaging Disposal Instructions	Can be recycled or incinerated.		

Instructions for use/application

Bladder protection briefs for moderate to heavy incontinence designed for highly demanding situations and for bedridden patients. Product size recognition by colour codes, names and numbering system. Always measure the hip size to select the right size and choose the correct absorption level to avoid leakages.

Fold the brief lengthways with the backsheet turned outwards to create a bowl shape. Pass the brief from front to back. Fan out the front of the brief, creating a channel shape between the legs. Fix the bottom tapes angled slightly upwards on both sides and then fix top tapes angled downwards for a secure and comfortable fit.

These products can be applied in upright position (both as self- management or assisted), and lying position. See our application guide brochure for detailed information.

Packaging data

Unit	Contains	Length	Width	Height	EAN
cll	96 pcs	57.6 cm	39.6 cm	36.7 cm	5713571000137
pck	24 pcs	38 cm	18 cm	26 cm	5713571000120



Nordic Swan Ecolabel is an official Scandinavian environmental label. Products with the Nordic Swan Ecolabel meet stringent environmental requirements with regard to raw material, production, use, documentation, and safety.



Your guarantee that our products have been tested for ingredients which can potentially cause skin irritation. Tests conducted by an independent third-party laboratory.



Regulation (EU) 2017/745 is a regulation of the European Union on the clinical investigation and sale of medical devices for human use. It repeals Directive 93/42/EEC, which concerns medical devices, and Directive 90/385/EEC, which concerns active implantable medical devices, on 26 May 2021.



The CE mark guarantees that a product is safe to use and complies with all safety precautions. CE stands for Conformité Européenne (European Conformity) and is mainly found on electronic equipment, safety equipment, construction products, and medical devices.



The UKCA mark guarantees that a product is safe to use and complies with all safety precautions in line with product standards and legislation of the UK.



The product does not contain latex.



Leakage barriers in the leg cut and around the absorbent core help prevent leakages and provide an optimal fit.



The product is fully breathable for the comfort of the user; also during longtime use. Helps maintain skin integrity.



The product has flexible tapes ensuring the best combination of comfort, fit, and leakage protection. The blue double tape guarantees an extra repositioning possibility.



The unique structure of our high performance core is designed for fast dispersion and encapsulation of urine, which results in a dry surface even after multiple voids.



Odour control is a critical factor in ensuring discreteness for the user. ABENA's incontinence products are designed and engineered to ensure that minimal odour arises from the product.





Unique Top-Dry Technology that provides a rapid absorption and ensures a dry surface even after multiple voids.



Drop system indicating absorption. The more drops, the higher the absorption. Ranges from 1 to 9 drops.



Wetness indicator is a liquidsensitive strip that changes colour or fades out indicating when it is time to change the product.



A combination of 100% breathability, Top-Dry and leakage protection provides a dry surface and healthy skin even after multiple voids maintaining the skin's natural pH balance.

ABENA





