



Item number 1000021282



Briefs, ABENA Slip, S4, yellow colourcode, Premium

- ✓ High performance core with Top-Dry Technology
- ✓ 360 ° leakage barriers preventing leakage all around
- ✓ Curved leg elastics for optimum fit.
- ✓ Wetness indicator with graduation scale
- ✓ Support healthy skin balance



Product Description

ABENA Slip is our Premium range of fully breathable, anatomically shaped bladder protection briefs for moderate to heavy incontinence. The high performance core with Top-Dry Technology ensures fast absorption and a dry surface even after multiple voids, maintaining skin integrity. A combination of different barriers (visible and invisible) and elastics (curved leg elastic) offer 360 degree leakage protection all around the product. When combined, these functionalities keep the skin in balance and maintains a natural pH skin balance. The odour system minimizes unpleasant smells.

The high quality flexible combi tapes with double tape function ensures an optimum fit and easy repositioning. The wetness indicator with graduation on scale guides the user and caretaker to the product with the right absorption level and aids product changes. The dermatological tested products feature the Nordic Eco Label.



Specifications

| | |
|----------------------------------|--|
| Base name | Briefs |
| Brand | ABENA |
| Sub-brand | Slip |
| Size | S4 |
| Color | White |
| Features | Yellow colourcode |
| Material | Breathable textile backsheet (BTBS), ECF fluff, elastane, elasticated combi tape, nonwoven, PE, PP, resin, SAP |
| Length/depth | 690 mm |
| Width | 520 mm |
| Hip size | 60-85 cm |
| Tolerance | +/- 10% |
| Absorption Rothwell | 2200 ml |
| Gender | Unisex |
| Quality level | Premium |
| Certifications | CE, MD, Nordic Swan Ecolabel, Dermatologically tested, UKCA |
| CE Class (Medical Devices) | Class I |
| Product or test standards | ISO 11948-1, ISO 13485, ISO 14001, ISO 9001 |
| Directives, regulations and acts | MDR (EU) 2017/745 |
| Safety Instructions and Warnings | Keep out of reach of children. |

| | |
|---|---|
| Because of the products composition it has an expiry date from the production date on: | 5 years |
| Storage Instructions | Store dry, room temperature and no direct sunlight. |
| Product Disposal Instructions | Dispose with household waste. |
| Packaging Disposal Instructions | Can be recycled or incinerated. |

Instructions for use/application

Bladder protection briefs for moderate to heavy incontinence designed for highly demanding situations and for bedridden patients. Product size recognition by colour codes, names and numbering system. Always measure the hip size to select the right size and choose the correct absorption level to avoid leakages.

Fold the brief lengthways with the backsheet turned outwards to create a bowl shape. Pass the brief from front to back. Fan out the front of the brief, creating a channel shape between the legs. Fix the bottom tapes angled slightly upwards on both sides and then fix top tapes angled downwards for a secure and comfortable fit.

These products can be applied in upright position (both as self- management or assisted), and lying position. See our application guide brochure for detailed information.

Packaging data

| Unit | Contains | Length | Width | Height | EAN |
|------|----------|---------|-------|---------|---------------|
| cll | 75 pcs | 70.6 cm | 40 cm | 18.7 cm | 5713571000076 |
| pck | 25 pcs | 38 cm | 18 cm | 22 cm | 5713571000069 |



Nordic Swan Ecolabel is an official Scandinavian environmental label. Products with the Nordic Swan Ecolabel meet stringent environmental requirements with regard to raw material, production, use, documentation, and safety.



Your guarantee that our products have been tested for ingredients which can potentially cause skin irritation. Tests conducted by an independent third-party laboratory.




Regulation (EU) 2017/745 is a regulation of the European Union on the clinical investigation and sale of medical devices for human use. It repeals Directive 93/42/EEC, which concerns medical devices, and Directive 90/385/EEC, which concerns active implantable medical devices, on 26 May 2021.




The CE mark guarantees that a product is safe to use and complies with all safety precautions. CE stands for Conformité Européenne (European Conformity) and is mainly found on electronic equipment, safety equipment, construction products, and medical devices.



The UKCA mark guarantees that a product is safe to use and complies with all safety precautions in line with product standards and legislation of the UK.

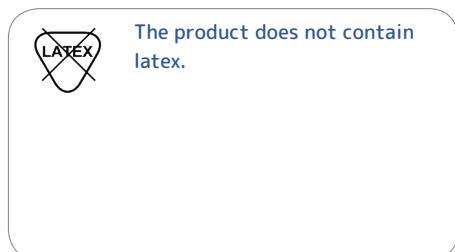

The product does not contain latex.

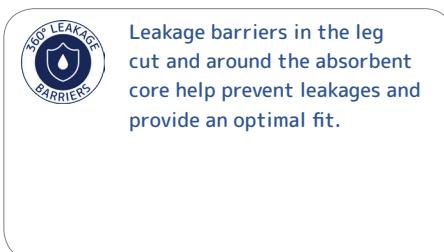


Leakage barriers in the leg cut and around the absorbent core help prevent leakages and provide an optimal fit.



The product is fully breathable for the comfort of the user; also during long-time use. Helps maintain skin integrity.



The product has flexible tapes ensuring the best combination of comfort, fit, and leakage protection. The blue double tape guarantees an extra repositioning possibility.

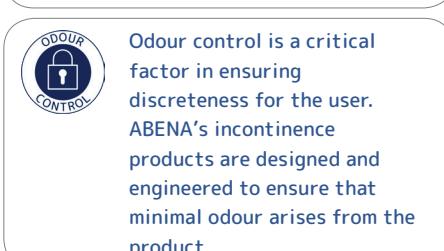

Odour control is a critical factor in ensuring discreteness for the user. ABENA's incontinence products are designed and engineered to ensure that minimal odour arises from the product.



Unique Top-Dry Technology that provides a rapid absorption and ensures a dry surface even after multiple voids.



Wetness indicator is a liquid-sensitive strip that changes colour or fades out indicating when it is time to change the product.



A combination of 100% breathability, Top-Dry and leakage protection provides a dry surface and healthy skin even after multiple voids maintaining the skin's natural pH balance.



Drop system indicating absorption. The more drops, the higher the absorption. Ranges from 1 to 9 drops.

