



Item number 254114

ABRI-SOFT
superdry

Underpad, Abena Abri-Soft Superdry, 180x70cm, light blue, with flaps

- ✓ Absorbent fluff and SAP core
- ✓ Soft and smooth nonwoven surface
- ✓ Rapid liquid absorption
- ✓ Liquid safe back sheet and sealed edges for extra protection
- ✓ Available in different sizes



Product Description

Abri-Soft Superdry is used as a protective, disposable underpad for e.g. beds and chairs. Suitable for incontinent users, but also for non-sterile procedures. The underpad contains both fluff and SAP, which encapsulates the liquid even faster and keeps the surface dry and comfortable. The odour system minimises unpleasant smells. In addition, the underpad has sealed edges for optimum leakage protection, and the surface is covered with a soft nonwoven material for enhanced user comfort. The use of disposable underpads reduces the risk of contamination.

Abri-Soft Superdry offers light to moderate absorbency and is available in different sizes. It is also available in a versions with flaps and a version with tape, which are usefull for active or restless users, as the underpad is fixated on the sheet.



Specifications

Base name	Underpad
Brand	Abena
Sub-brand	Abri-Soft Superdry
Sub-color	Light blue
Properties	With flaps
Features	Non-sterile, each side flap measures 70x50 cm
Single or multiple use	Single use
Material	Fluff, nonwoven, PE, SAP, tissue
Length/depth	180 cm
Width	70 cm
Weight, net	140 g
Tolerance	+/- 2,5 cm
Sterilization method	EO
Absorption Rothwell	1750 ml
Certifications	CE, Dermatologically tested, FSC Mix, MD
CE Class (Medical Devices)	Class I
Product or test standards	ISO 11948-1
Directives, regulations and acts	MDR (EU) 2017/745
Because of the products composition it has an expiry date from the production date on:	5 years
Storage Instructions	Store dry, room temperature and no direct sunlight.

Product Disposal Instructions

To be disposed of with ordinary household waste. If contaminated, dispose of as clinical waste.

Packaging Disposal Instructions

Can be recycled or incinerated.

Instructions for use/application

The underpad must be placed on the bed or chair with the blue side down. Ensure that the underpad is fully smoothed out before use.



Regulation (EU) 2017/745 is a regulation of the European Union on the clinical investigation and sale of medical devices for human use. It repeals Directive 93/42/EEC, which concerns medical devices, and Directive 90/385/EEC, which concerns active implantable medical devices, on 26 May 2021.



FSC™ (Forest Stewardship Council) is an international non-governmental organization dedicated to the promotion of responsible forest management worldwide. FSC™-certified products are made from raw materials from well-managed forests and responsible sources. The label's traceability and integrity are upheld through stringent control measures, and by choosing FSC™-certified products, you help take care of forests and the people and wildlife that call them home. FSC™ Mix means the product consists of a blend of FSC certified wood and recycled materials. For more information, go to www.fsc.org.



The Dermatologically Tested symbol guarantees that products have been tested for potential skin irritations by an independent third-party laboratory.



The CE mark guarantees that a product is safe to use and complies with all safety precautions. CE stands for Conformité Européenne (European Conformity) and is mainly found on electronic equipment, safety equipment, construction products, and medical devices.



The product is only for single-use.



Raw material, polyacrylate, that absorbs its own weight 33 times.



Abena's incontinence products are designed and engineered to ensure minimal odor arises from the product. Odor control is a critical factor in ensuring discreteness for the user.



Raw material that absorbs its own weight 10 times and has excellent fluid dispersion capabilities.